	BAAT MEDICAL
	BAAT Medical Products B.V. F. Hazemeijerstraat 800 7555 RJ Hengelo The Netherlands
	Phone: +31 (0)88 565 66 00 www.baatmedical.com
	CE 0482
	12027BR190114 2018-01-14

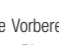
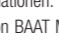
ENGLISH
<p>Important Medical Information</p>
<p>PRE-OPERATIVE PRECAUTIONS</p>
<p>This package insert includes information on basic precautions and instructions for single use sterile medical devices from BAAT Medical Products. Please consult the applicable surgical technique for intended use, indications, contraindications, selection and use of a device and check the full labeling for other necessary information. Surgical technique brochures may be by requested from the distributor or from BAAT Medical Products directly. Those using brochures published more than two years before the surgical intervention are advised to obtain an updated version.</p>
<p>BAAT Medical Products devices can only be used by surgeons who are fully familiar with the surgical technique required and who have been trained to this end. The operating surgeon must take care not to use the specialized instruments to exert inappropriate stress on the patient or the implants and must scrupulously comply with any operating procedure described in the surgical technique provided. For example, the forces exerted when repositioning an instrument in-situ must not be excessive as this is likely to cause injury to the patient.</p>
<p>To reduce the risks of breakage, care must be taken not to distort the implants or nick, hit or score them with the instruments unless otherwise specified by the applicable surgical technique. Extreme care must be taken when the devices are used near vital organs, nerves or vessels.</p>
<p>CONTACT</p>
<p>See brochure for telephone and address of local representative/distributor, or contact BAAT Medical Products.</p>
<p>STERILITY</p>
<p>Devices which are delivered sterile have been exposed to a minimum of 25kGy of gamma radiation from a cobalt 60 source.</p>
<p>To reduce the risk of infection the packaging of all sterile devices must be inspected for flaws in the sterile barrier or expiration of shelf life before opening. In the presence of such a flaw or expiration of shelf life, the device is considered non-sterile, and must be discarded.</p>
<p>Do not resterilize. Devices not labeled as sterile are non-sterile.</p>
<p>SINGLE USE</p>
<p>The implants are single use only.</p>
<p>EXPLANATION OF SYMBOLS</p>

LOT	Batch number		Do not resterilize
REF	Catalogue number		Manufactured by
	Single-use		Use by date
	Consult instructions for use	QTY	Quantity
STERILIZED	Sterilized using irradiation		Keep dry
	Do not use if package is damaged		Caution - consult instructions for use
	Upper limit of temperature is 45°C		

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